Because every scar needs care!

# Scarban

Professional Scar Management

## Silicone sheets

Instructions for use

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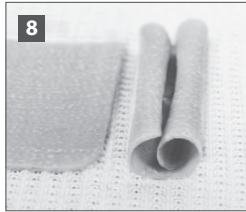
25340002

- This is necessary to ensure good adhesion.
- Rub the silicone sheet with a few drops of Scarban Silicone Cleaning Soap or a neutral soap (fig. 6).
- Remove as much dirt as possible with soap and lukewarm water.
- Rinse the silicone sheet thoroughly under lukewarm running water (fig. 7).
- After contact with soap and water, the silicone sheet will no longer be adhesive and it will roll up (fig. 8).
- Leave the silicone sheet to dry on a clean, dry surface, with the sticky side facing upwards (fig. 8).
- While drying, the silicone sheet will unroll and its adhesive strength will gradually return.
- Use the silicone sheet only after it has completely dried.
- Trim edges of silicone sheets that are no longer adhesive, due to wear and tear.

- How long should I use Scarban silicone sheets?**
- It takes at least 3-6 months for a scar to normalise.
  - Use the Scarban silicone sheet as long as the scar is raised, red or causes complaints.
  - Consult your physician about the duration of the therapy.

- Duration of use**
- Each pack of Scarban contains one or more silicone sheets.
  - If used correctly and cared for properly (according to the instructions for use), each pack of Scarban is sufficient for approximately 4 to 8 weeks of treatment.
  - In the event of long-term therapy, several packs may therefore be necessary.

- Special attention and precautions**
- Keep out of reach of children.
  - Scarban silicone sheets are only suitable for external use.
  - Use Scarban silicone sheets only on healed, closed skin (after the scab has disappeared) and never on open wounds.
  - Never dry Scarban silicone sheets in the sun or near a heat source.
  - Temporarily remove the Scarban silicone sheet when exercising or swimming.
  - Perspiration or contact with water may cause the Scarban silicone sheet to detach.
  - Do not use if you are hypersensitive to any of the ingredients.
  - Do not use if the packaging is damaged when you receive the product.

- Side effects**
- In rare cases, the skin may have a hypersensitive reaction (skin irritation) to silicone treatment. In such cases, stop the treatment until the skin is no longer irritated.
  - Start the treatment again and let the skin slowly get used to the Scarban silicone sheet (applying it for an additional hour every day).
  - If skin problems persist, consult your physician.
  - If there are serious adverse effects related to the product, these should be reported to the manufacturer and to the competent authorities (of the Member State) where the user is located.

- Storage**
- The expiry date is stated on the packaging.
  - Scarban silicone sheets can be stored for 3 years in its unopened packaging.
  - Store at temperatures between 5 and 25°C.

**Date of latest revision of instructions for use**  
June 2024

Manufacturer's name: Scarpro N.V. Lobulckstraat 3, Alter, 9880, Belgium.  
License holder: Biomed JR Ltd. PO Box 3728 Tel Mond  
Distributor: Novolog Ltd. Ha'mayan 55 Modi'in  
Registration number in AMR is 25340002

## Scarban silicone sheets - Elastic and Light

**Product Description**  
- Scarban silicone sheets are a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone sheets glove and finger ), washable and can be used for a long period of time.  
- Scarban Silicone Cleaning Soap is a cleaning agent for silicone sheets.

**Composition**  
- Textile layer: polyamide, elastane  
- Medical grade silicones

**Properties**  
- Scarban silicone sheets ensure that scars become flatter, smoother and thinner and reduce complaints (like persistent redness, pulling of the scar , itching, restriction of joints movement, discomfort and cosmetic concerns).  
- The silicone sheet provides a semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration ensures that the damaged scar skin recovers more quickly and that functional objections (complaints) are reduced.  
- Scarban silicone sheets provide UV protection as standard – at least 93% UV blocked.

**Indications/applications**  
Scarban silicone sheets are suitable for:  
- both old and new scars  
- all skin types  
- all ages  
- treatment and prevention of abnormal scars (hypertrophic and keloid)  
- treatment of burn scars  
- treatment of surgical scars  
- use under clothing and in compression therapy  
- treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar , itching, restriction of joint movement, discomfort and cosmetic concerns)

**Instructions for use – Scarban silicone sheets**  
- For optimum results preferably start using Scarban silicone therapy immediately after wound closure (after removal of sutures and the disappearance of any scabs).  
- It is recommended that Scarban silicone therapy is build up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.  
- For the best possible results, it is preferable that the silicone sheets are used for a minimum of 12 and a maximum of 23 hours per day.  
- If necessary, Scarban silicone sheets can be cut to the correct size (fig. 1).  
- Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.  
- Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar site.  
- Thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding it (fig. 2).  
- Remove the protective film from the back of the silicone sheet (fig. 3) and apply Scarban to the scar (fig. 4).  
- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).  
- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

**Care instructions for Scarban silicone sheets**  
- It is essential to clean the Scarban silicone sheets properly, on a daily basis.

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